
Tesmed+

absolute™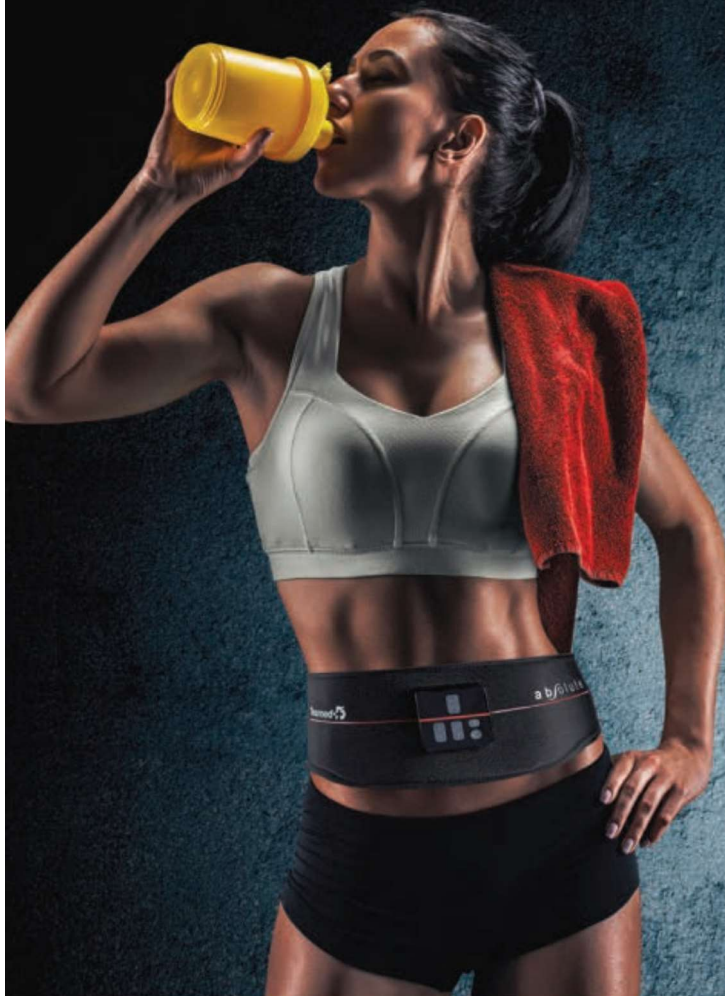

ITALIANO

ENGLISH

DEUTSCH

FRANÇAIS

ESPAÑOL

PORTUGUÊS

POLSKIE

-  **Manuale d'uso**
-  **Instruction manual**
-  **Mode d'emploi**
-  **Gebrauchsanweisungen**
-  **Manual de instrucciones**
-  **Manual de instruções**
-  **Instrukcja obsługi**

Made in Italy

Gentile Cliente,

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Dear Customer,

*Congratulations on your purchase of Tesmed **absolute**®.*

Our name is a guarantee of high quality products and subjected to regular checks. In case of information and / or assistance, please write to info@tesmed.com, we will try to answer you as soon as possible.

Please read all of your user manual before using this product and comply with the warnings given.

Thank you

*Best Regards
Tesmed*

© Partial and total duplication is prohibited by law - www.tesmed.com

ITEMS CONTAINED

- 1 Control device (*pre-positioned on the belt*)
- 1 Belt
- 1 USB cable for charging
- 1 Nebulizer
- 1 User manual
- 1 Warranty information

CONTENTS

General information	page	27
High technology	page	27
Symbols used	page	28
Warnings	page	28
Precautions.....	page	29
Possible side effects.....	page	29
Contraindications.....	page	29
Maintenance	page	31
Operating instructions.....	page	32
Description of the device	page	32
Preparation of tesmed absolute.....	page	33
Detaching the device from the belt.....	page	33
Adjusting the belt size.....	page	33
Moistening the abdomen and conductive pads.....	page	34
Application of the belt on the abdomen.....	page	35
Two completely separate channels	page	35
Switching on	page	35
Programme selection.....	page	35
Adjusting intensity - 99 levels	page	36
Error detection	page	36
Keypad lock.....	page	36
Pause	page	36
Rapid overview of programmes.....	page	37
Switching off	page	37
Storage.....	page	37
Recharging the battery	page	38
Charging time	page	38
Body position during stimulation	page	38
Suggested training scheme and choice of programmes	page	39
Tesmed Absolute - Programme description.....	page	41
Troubleshooting guide	page	43
Notes regarding the battery	page.	44
Disposal.....	page	44
Technical characteristics of absolute	page	45
Contacts	page	45
Declaration of conformity.....	page	155

GENERAL INFORMATION

Tesmed absolute is an electrostimulation belt for the abdominal muscles that uses electric muscle stimulation (EMS) technology. The central unit transmits bioelectric impulses through the skin with the aim of strengthening the muscles; the impulses cause the muscle fibres to contract and relax.

By using **Tesmed absolute** 5 times a week, in just 6 to 8 weeks, you can expect the following results:

- A more toned abdomen.
- More defined muscles.
- Better trained muscles in the treated area.
- A feeling of relaxation of the muscles involved.

For best results, we recommend using the device in combination with a normal, healthy diet and exercise.

HIGH TECHNOLOGY

Tesmed absolute is designed and built with cutting-edge electronic components, a high storage microcontroller with great memory, processing and storage capacity. This means that the **Tesmed absolute** is an extremely professional product with superior performance, yet compact size.

SYMBOLS USED

Important note, if not respected, there may be a risk for the user and/or the equipment may be damaged. We recommend that you read the information in this manual carefully and keep it in a safe place.

Low frequency parts applied.

Internally fed equipment (DC).

Parts that must be disposed of as special waste.

Read the operator's manual

Manufacture

Double electrical isolation

Protection level

IP41

This symbol on the device indicates that the belt is protected against solid bodies of a dimension of more than 1 mm in diameter and against drops of water falling vertically from above.

WARNINGS

- You must take into account the technical characteristics of each program and read the relevant paragraphs in the User Manual.
- Tesmed absolute must not be used for purposes other than those indicated.
- You must begin each treatment with the current emission value set at zero (min channel power setting) and then gradually increase and/or reduce the power to suit your tolerance.
- Max power does not necessarily mean better performance or results.
- Tesmed absolute must be kept out of the reach of children.
- Electrocardiogram monitoring equipment may be disturbed by the current used for the electro stimulator.
- We do not recommend using Tesmed absolute on children under the age of 14 or the elderly over 75.
- Do not use the equipment if there are special health problems without first obtaining medical advice from a doctor.
- Should the equipment malfunction, do not try to open or repair the equipment, but send the product in its original packaging to FELDI SRL, describing the problems found on the inside of the package (see warranty conditions on www.tesmed.it).
- Device not suitable for simultaneous use with high frequency surgical devices
-  The application of electrodes close to the chest may increase the risk of heart fever.

- The electrodes should never be placed on the carotid breast, along the head, directly on the eyes, covering their mouths and / or on a spastic muscle.

PRECAUTIONS

- Do not tamper with the equipment or try to open it.
- Keep the electro stimulator away from sources of heat, powder, sprayed water, humid atmospheres and avoid direct exposure to the sun.
- Avoid violent knocks that could damage the equipment.
 Operating temperature: 0 °C 45 °C
 Storage temperature for a period of 1 month: -20 °C 45 °C
 Storage temperature for a period of 6 months: -10 °C 35 °C
 Humidity: between 20 and 90% RH (without condensation)
 Atmospheric pressure: between 75 and 106 kPa

As for the central unit, the manufacturer guarantees, with correct use/maintenance, a useful life of the equipment of 2 years.

If, on the other hand, the belt shows signs of wear, it may need to be replaced. The belt is to be considered a spare part.

POSSIBLE SIDE EFFECTS

- Isolated cases of skin irritation may occur in people with highly sensitive skin. Should there be an allergic reaction to the **Tesmed absolute**, stop the treatment and consult a specialist doctor. It is normal for the skin to be slightly red after the treatment in the areas where the electrodes were placed: this will disappear after roughly 30 minutes. We strongly advise against treating any areas of the body with serious skin problems.

CONTRAINDICATIONS

ATTENTION:

Electro stimulation is not recommended in areas with any of the following:

- inflammation, open wounds, strained or torn muscles, dermatitis, bacterial infections, erythema or skin irritation;
- muscular hernias;
- fractured bones;
- varicose veins or vascular disease in general;
- neoplasia;
- mucous membranes.
- in the thoracic area to people with cardiac arrhythmia.

This device is prohibited in the following cases:

- in pregnant women;
- in patients fitted with a pacemaker (risk of interference with pacemaker activity) and/or an implantable defibrillator;
- in cases of peripheral nerve disorders, muscle diseases, and severe damage to osteoarticular structures;
- in severe heart diseases;
- in patients suffering from epilepsy;
- oncological diseases;
- severe arterial hypertension;
- in individuals with a significant ongoing systemic, debilitating, or dermatological condition;
- in patients undergoing treatments with HF (high-frequency) equipment, as this may cause burns and damage to the electrostimulator itself;
- in the vicinity of shortwave or microwave therapy equipment, as this may cause instability in the electrostimulator output;
- in patients who are unconscious or unable to communicate;
- in individuals whose psychophysical conditions do not allow exposure to external factors;
- in the presence of a flammable anesthetic mixture with air or oxygen or nitrous oxide;
- if the device must be worn over drug/medication injection sites (for short or long periods), for example during hormonal therapies;
- acute diseases;
- in cases of a tendency to bleeding, e.g. after acute injuries or a fracture. The stimulation current may cause or worsen bleeding;
- after surgical operations where stronger muscle contractions could interfere with the healing process;
- high fever or psychosis;
- in the presence of acute or chronic diseases of the gastrointestinal system, including inflammatory or undiagnosed abdominal conditions such as diverticulitis.

Please wait before using product until:

- At least six weeks after the birth of your baby (you must consult your doctor first).
- At least three months after having a Caesarean (you must consult your doctor first).
- One month after an IUD contraceptive device (e.g. coil) has been fitted.
- The heavy days of your period have finished.

Please get your doctor's advice before using your product if:

- You wish to use belt and have a bad back.
- You have any serious illness or injury not mentioned in this guide.
- You have recently had an operation.
- You take insulin for diabetes.
- You want to use it on a young child.
- You suffer from muscle or joint problems.

General safekeeping:

- After strenuous exercise or exertion always use a lower intensity to avoid muscle fatigue.
- Do not use your controller at the same time as any other device which transfers an electrical current into the body (e.g. another muscle stimulator).
- Cease using your product if you are feeling light header or faint. Consult your doctor if this happens.

- Do not use while driving, operating machinery or cycling.
- For hygiene reasons the belt is for use only by one person. Do not share your belt with anyone else.
- Do not over exert yourself when using muscle stimulation. Any workout should be at level comfortable for you.
- Do not use your product if you are wearing a belly-button ring. Remove the ring before you begin session.
- Although compliant with applicable EMC requirements, this device may still interfere with more sensitive equipment, please move away or switch off.

If you are in any doubt about using your belt for any reason, please consult your doctor before use.

ATTENTION: Never place the electrodes on the carotid artery and/or on a spastic muscle. Use of the Tesmed absolute for therapy purposes should be supervised and recommended by a doctor. Interrupt application immediately in the event of discomfort and obtain medical advice.in the thoracic area to people with cardiac arrhythmia.

MAINTENANCE

Tesmed absolute is a highly sophisticated product and must be handled with care. The maintenance tips below will help you ensure its long life.

- Avoid getting the device wet. Rain, humidity, liquids, or condensation generally contain corrosive mineral substances that may damage electronic circuits.
- Keep Tesmed Absolute out of the reach of children.
- Do not leave Tesmed Absolute in particularly hot environments. Excessively high temperatures may reduce the lifespan of electronic circuits, damage the battery, and deform plastic parts.
- Do not leave Tesmed Absolute in particularly cold environments, as during use it may warm up and cause condensation to form inside, potentially damaging the electronic circuits.
- Do not open Tesmed Absolute. Any intervention carried out by unqualified personnel may cause damage.
- Do not drop, knock, or shake the device, as the internal electronic circuits may suffer irreparable damage.
- Do not use the battery for purposes other than those specified.
- Never use damaged batteries or chargers.
- Do not short-circuit the battery.
- Use only the supplied charger. In the event of malfunction, contact customer service.
- Do not leave Tesmed Absolute in hot or cold environments, such as inside a closed car during summer or winter, as this reduces battery capacity and lifespan.
- Do not dispose of Tesmed Absolute in fire!
- Dispose of Tesmed Absolute in accordance with local regulations, for example by recycling it. Do not dispose of it with household waste.
- To clean the Tesmed Absolute belt, remove the central unit and use a soft, slightly damp cloth that does not contain any solvents. Solvents may damage plastic parts.
- Never wash the belt in a washing machine or by hand.
- Do not use bleach to clean the belt.
- Do not dry-clean the belt.
- Do not dry the belt in a tumble dryer. Allow it to dry flat on a level surface. Do not dry it on hot objects or surfaces (such as a radiator), as the belt contains plastic parts. Ensure that the belt is completely dry before using it again.
- Do not iron the belt.

As regards the central unit, the manufacturer guarantees, with correct use and maintenance, a service life of 2 years. The belt, on the other hand, may need to be replaced if signs of wear appear. The belt is considered a replacement (spare) part.

OPERATING INSTRUCTIONS

DESCRIPTION OF THE DEVICE

- 1** ON/OFF and pause button
- 2** Programme and keypad lock button
- 3** All-channel intensity adjustment button
- 4** Central-channel intensity adjustment button
- 5** Side-channel intensity adjustment button
- 6** Timer symbol
- 7** Battery indicator
- 8** The two solid central rectangles represent the central conductive plates
- 9** Level of intensity for the central conductive plates
- 10** The two solid lateral rectangles represent the lateral conductive plates
- 11** Level of intensity for the lateral conductor plates
- 12** Programme number
- 13** Time remaining for the selected programme in minutes
- 14** Programme symbol
- 15** Keypad lock symbol
- 16** Battery recharge

PREPARATION OF TESMED ABSOLUTE

DETACHING THE DEVICE FROM THE BELT

The control device is attached to the belt with four magnets. During the belt preparation, it is recommended to detach it to avoid accidental activation.

To detach it, hold it between your fingers and pull with some force to overcome the magnetic attraction (**Photos 1, 2, 3**).

ADJUSTING THE BELT SIZE


First of all, you need to **test the size of Tesmed Absolute**. Generally, it fits well for medium and large waist sizes. If you have a small waist, adjust it by following the steps (**Photos 4, 5, 6**).

MOISTENING THE ABDOMEN AND CONDUCTIVE PADS

Take a sheet of kitchen paper towel (e.g., Scottex), fold it multiple times (**Photo 7**), wet it under running water (**Photo 8**), and apply it to your abdomen (**Photo 9**) to moisten the area to be treated.

Moistening the abdomen is not necessary if the pads are properly moistened.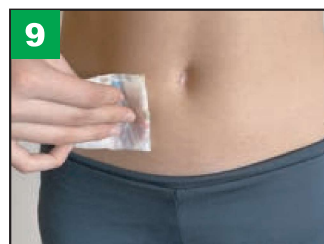

⚠ NEVER USE TESMED ABSOLUTE WITHOUT MOISTENING THE PADS

Place the Tesmed Absolute belt **upside down** on a flat surface and spray it with the **nebulizer (Photo 10)** evenly to properly moisten the entire area of each individual conductive pad (**Photo 11**).

⚠ Warning: Do not place the belt under running water and do not immerse it in water.

⚠ Make sure that the surfaces of the conductive pads are well and evenly moistened to avoid voltage peaks in specific points of the skin during the session.

(The surface of the pads is designed to easily retain a sufficient amount of water to create optimal conductivity with the skin, allowing up to two consecutive treatments without the need to moisten again.)

APPLICATION OF THE BELT ON THE ABDOMEN

⚠ Before starting each session, make sure that the skin is clean and free from creams or oils. Only in this way can the device function correctly.

Wear the belt as specified below.

Fasten the belt around the abdomen, ensuring **good contact between the conductive pads and the skin (Photos 12, 13).**


The navel should be centrally covered.

Attach the control unit to the belt (**Photos 14, 15**).

TWO COMPLETELY SEPARATE CHANNELS

Two separate channels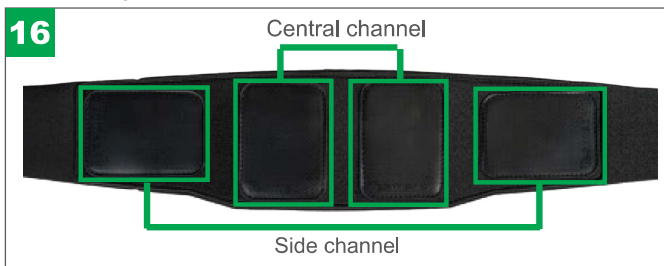

Since the hip area requires a different level of intensity than the central abdomen, the Tesmed absolute belt has been designed with 2 totally separate channels (**Photo 16**) so that the side muscles are completely separated from the central ones in order to adapt the intensity on the central and side parts of the abdomen appropriately.

You can also choose to focus treatment exclusively on the central or lateral part of the abdomen. **Tesmed absolute** can be used by people with very different waist sizes and allows more satisfactory stimulation to be achieved without discomfort as the overall electrical conduction area is particularly extensive.

SWITCHING ON

Briefly press the **ON/OFF** button to switch on the control unit.

Each time the device is switched on, it will be set to program 1.

PROGRAM SELECTION

Briefly press the **P** button to select one of the 15 programmes.

Each time the **P** button is pressed, it switches to the next programme.

If you want to switch to another programme while a programme is running:

- a** - if the keypad lock has previously been activated, press the **P** button for 2 seconds to unlock it, then briefly press the **P** button; intensity is reset, and the device automatically switches to the next program.
- b** - if the keypad lock has not been activated, briefly press the **P** button, intensity is reset, and the device automatically switches to the next program. 5 seconds after the program has been selected, the **P** on the display will be replaced by the battery status.

Once the program has ended, the device will reduce intensity to zero, while remaining on the program just completed.

If no action is taken once the programme is complete, the device will switch off after 30 seconds.

ADJUSTING INTENSITY - 99 LEVELS

Adjusting the intensity on **Tesmed absolute** is very practical.

There is a button for adjusting the intensity for each channel as well as a practical two-arrow button that allows you to adjust the intensity of both channels simultaneously.

Once the optimal intensity has been reached via the button with two arrows, it is possible to use one of the two buttons for an individual channel (indicated with a single arrow) to fine tune the setting. Once done, intensity can once again be adjusted using the two-arrow button; the difference in intensity between the two channels will be automatically maintained.

It is common, and completely normal, that at the same intensity on both channels, stimulation on the outer part of the abdominals is perceived as lower; it is therefore possible to selectively increase intensity with the single button.

The central **single button** adjusts the two central inner electrodes, and the **side single button** adjusts the intensity of the two side outer electrodes, eliminating the possibility of confusion.

To quickly increase or decrease intensity, press and hold the button for 2 seconds and adjustment will take place rapidly. This applies to both the single, one-channel button, and the double, two-channel button.

IN ORDER TO ACHIEVE THE DESIRED MUSCLE CONTRACTION, THE LEVEL OF INTENSITY MAY VARY DUE TO A NUMBER OF FACTORS, INCLUDING THE POSITIONING OF THE BELT ON THE ABDOMEN AND THE PRESENCE OF ADIPOSE TISSUE.

THEREFORE, THE SAME INTENSITY OF CURRENT CAN PROVIDE DIFFERENT SENSATIONS FROM PERSON TO PERSON, FROM DAY TO DAY, FROM LEFT TO RIGHT, AND SO ON.

ERROR DETECTION

If the control device is not properly positioned in its slot, or if the belt is not well adhered to the skin, or if the electrodes and skin are dry, or if there are micro-residues on the button contacts, **the device will enter an error state, emitting an acoustic signal and displaying a message on the screen. In this case, check the aspects mentioned above and refer to page 43, "Troubleshooting".**

KEYPAD LOCK

Pressing and holding the P button **for 2 seconds** will activate the **key lock** function **and the key symbol will appear on the display in place of the T for Time.** The device can still be switched off by pressing and holding the **ON/OFF** button for 2 seconds, even with the key lock activated.

PAUSE

By *briefly* pressing the **ON/OFF** button, the device will pause, momentarily interrupting stimulation and stopping the timer, and the blinking square on the display, showing the minutes, will disappear.

By pressing the **ON/OFF** button again, the device will resume operation with the same intensity as before and the square showing the minutes will start to flash again. Pause can also be activated and deactivated even when the key lock key symbol is showing on the display.

ATTENTION: If you leave the device paused for several minutes, when stimulation recommences, intensity may seem excessively high for a few minutes. The device can also be paused by reducing intensity to zero on both channels; again, in this case the timer will stop.

In this case, if you try to reactivate stimulation with the **ON/OFF** key, the device will not restart and will remain at zero, unless, in the same Tesmed absolute session, the **ON/OFF** key has been used at least once to pause, in which case the device will restart with the most recent intensity stored when the pause function was used. AFTER 20 MINUTES of continuous pause, the device will switch off.

RAPID OVERVIEW OF PROGRAMMES

By using the pause ON/OFF button, you can quickly explore all 15 programmes without having to increase the intensity every time from zero.

The programmes can also be explored in this manner:

For example, you can start with programme 1, at a comfortable, rather low intensity, for example 10; press the pause button to pause the device and memorise intensity 10, then press the **P** button to go to the next programme, which is 2 (*when you change the programme the device will always go to zero intensity*), then press the pause button to reactivate stimulation, the device will immediately start with intensity 10.

At this point you can directly press the **P** button to switch to the next programme, in this case 3, and press the pause button, at which point programme 3 will start directly at intensity 10 and so on.

ATTENTION: despite starting at the same intensity, some programmes will be perceived as stronger and others as weaker, due to their different structure. Therefore, it is advisable to use this function of rapid programme exploration at a low intensity.

Turning the device off and on again will reset the pause button memory.

SWITCHING OFF

To turn off the device, press and hold the **ON/OFF** button for 2 seconds.

STORAGE

When storing Tesmed absolute between sessions, it is not necessary to detach the control device from the belt, but it is recommended that the device be stored in a safe place and out of the reach of children.

When storing without use for long periods, it is recommended that **Tesmed absolute** be stored with a battery charge level of between 30% and 50%.

RECHARGING THE BATTERY

The battery can be charged either with the control device attached to the belt or detached, as desired.

Connect the supplied cable, with the micro-USB Type-C end inserted into the control device (**Photo 17**) and the other end with USB to a mobile phone charger (**Photo 18**).

When charging, the symbol will flash showing the current charge (**Photo 19, 20**), and will remain steady when the battery is fully charged (**Photo 21**). The battery charge indicator will remain visible until disconnected from the charger.

CHARGING TIME

A full charge can take **3 to 4 hours**, depending on the charger you use.

A full charge will allow between 4 and 50 treatments:

the number depends on the intensity of the treatment and the type of programme selected.

For example: programme 1 at low intensity consumes the least amount of power, while programme 10 at maximum intensity consumes the most power.

Even at the same intensity, for example, due to its characteristics programme 10 consumes much more power than the other programmes.

BODY POSITION DURING STIMULATION

Stimulation with **Tesmed absolute** can be performed while sitting or standing, except for the first treatment, which must be performed while sitting, as specified in the “WARNINGS” section.

SUGGESTED TRAINING SCHEME AND CHOICE OF PROGRAMMES

The programmes are in training order from 1 to 15; generally, a beginner starts with programme 1 and reaches programme 15 in about 2 months.

For the more experienced, for maintenance training, programmes 10 to 15 are recommended.

If used at a comfortable intensity, the programmes usually provide a pleasant sensation as long as the conductive plates and the abdomen are well moistened, but if any programme is uncomfortable, it can be replaced with a programme of your choice other than the one indicated in the proposed training scheme below.

The user can create a personalised training plan based on the preferred programme or programmes. The number of sessions is indicative, but between 3 and 5 sessions per week can generally be carried out as preferred. It is advisable not to use the device for more than one hour per day. If you want to do **two sessions on the same day, it is advisable to wait at least 5 hours after the first one.** Always leave one or two days off per week.

For example, an **8-week** training schedule is shown below. **This schedule can be customised by reducing the number of programmes and/or sessions recommended for each week, if preferred, trying to do at least 3 sessions per week in total.**

In some cases, especially in the presence of a narrow waist, lateral stimulation may be less tolerable, in which case it is suggested to lower the intensity, or trying a change of position (e.g., standing instead of sitting) and/or to activate the intensity only on the central stimulation thus leaving the lateral stimulation inactive.

Tesmed absolute can be used standing, sitting or lying down. More experienced users, followed by the athletic trainer or personal trainer, can use Tesmed absolute during physical exercise.

PROPOSED 8-WEEK TRAINING SCHEME FOR 5 DAYS A WEEK		
Week	Progr.	Sessions
FIRST	1	Once a day for the first 3 days
	3	Once a day for the successive 2 days
SECOND	2 and 3	Both programmes once a day for 5 days
THIRD	4	Once a day for the first 3 days
	5	Once a day for the successive 2 days
FOURTH	5	Once a day for the first 3 days
	6	Once a day for the successive 2 days
FIFTH	7	Once a day for 5 days
SIXTH	8 and 9	Both programmes once a day for the first 3 days
	9	Once a day for the successive 2 days
SEVENTH	10 and 11	Both programmes once a day for the first 3 days
	12	Once a day for the successive 2 days
EIGHT	13 and 14	Both programmes once a day for the first 2 days
	15	Once a day for the successive 3 days

At the end of the 8-week cycle, you can continue maintenance training with one session per day alternating 3 days per week, using a programme of your choice from 10 to 15.

The training scheme can also be chosen by the user according to their state of well-being and/or their preferences, as well as the duration in weeks of the treatment cycle.

For a more intensive workout, you can also use just programmes 14 and 15, even a session of both on the same day as long as you wait at least 5 hours between programmes, for no more than 5 days per week.

Following treatment, temporary itching or redness of the treated area may occur due to the effects of stimulation.

TESMED ABSOLUTE - PROGRAMME DESCRIPTION

Toning programmes - basic level

PROG. 1 - duration 25 minutes

Initial level - a programme with a single phase of stimulation, low frequency, ideal for those using EMS stimulation for the first time.

PROG. 2 - duration 25 minutes

Intermediate level - a programme with a single phase of stimulation, but with a higher frequency than the previous programme, lasting for a longer period. Suitable for beginners gaining confidence with EMS stimulation or for those returning to training with the belt following a long period of inactivity.

Progressive toning programmes

PROG. 3 - duration 25 minutes

This programme marks the passage from a single phase of stimulation to progressive toning. The frequency is higher than the preceding programmes. Complete the training session before passing on to the following programme.

PROG. 4 - duration 25 minutes

This is a programme with a single phase of stimulation that is more challenging than the previous programmes; the frequency is higher, as is the number of impulses. If possible, gradually increase intensity.

PROG. 5 - duration 25 minutes

This programme, with a single phase of stimulation, completes the third week of progressive training. If you are able, increase intensity to a point at which stimulation is pleasant yet vigorous.

Optimisation programmes - can be added as a continuation of the training programme or used as you like

PROG. 6 - toning, definition and strength training - duration 30 minutes

A programme with 10 different phases of stimulation that alternate in a cycle with variable frequency, of which 5 are in close sequence (each impulse perceived consists of 6 indistinguishable impulses in rapid succession). Useful for maintaining and increasing abdominal strength and definition.

PROG. 7 - toning and strength training - duration 30 minutes

An advanced programme for abdominal activation; the toning effect is powerful, and is aimed at increasing strength. Two different phases of stimulation alternate in a cycle; the first is at a variable frequency of up to 120 Hz, while the second is an intense series of impulses (each impulse perceived consists of 6 indistinguishable impulses in rapid succession).

This programme completes the fifth week of training.

Additional training programmes - can be added as a continuation of the training programme or used as you like

PROG. 8 - toning and strength training - duration 30 minutes

A programme that alternates 3 different phases of stimulation in a cycle, aimed

at obtaining stronger and more toned abdominal muscles. The first two different phases are at a variable frequency of up to 150 Hz, while the third is an intense series of impulses (each impulse perceived consists of 6 indistinguishable impulses in rapid succession). During this programme, strong and prolonged contractions alternate with more incisive and rapid stimulation.

PROG. 9 - toning, definition and firming - duration 30 minutes

This programme is ideal for defining and firming the abdomen. It consists of an alternating cycle of 6 different phases of stimulation, of which 3 are in close sequence and each of a higher frequency than the previous (each impulse perceived consists of 6 indistinguishable impulses in rapid succession).

PROG. 10 - toning, strength training and resistance - duration 30 minutes

A programme with a single phase of stimulation. You can use this programme to increase the strength and resistance of abdominal muscles. It consists of a single intense phase interspersed with extremely brief pauses. Choose a comfortable level of intensity.

PROG. 11 - toning, definition and strength training - duration 30 minutes

An advanced programme with 12 different phases of stimulation that alternate cyclically, with variable and increasing frequency in the first 7 phases, decreasing over the remaining 5 phases. This is a complete programme that is useful for muscle tone, definition and strength.

PROG. 12 - toning, definition and strength training - duration 30 minutes

An advanced programme with 10 different phases of stimulation that alternate cyclically, with variable and increasing frequency in the first 6 phases, decreasing over the remaining phases; some phases are at a higher frequency than the previous programme. This is a complete programme that is useful for muscle tone, definition and strength.

Advanced alternating stimulation programmes; can be used as a continuation of the training programme or at your discretion

PROG. 13 - toning, definition and strength training - duration 30 minutes

An advanced programme that alternates stimulation of the central and lateral abdominal muscles autonomously and progressively, with a series of impulses at 60 Hz that alternate every 5.5 seconds. Muscles are stimulated in alternation for an intense and continuous training that tones, defines and increases strength.

PROG. 14 - toning, definition and strength training - duration 30 minutes

An advanced programme that alternates stimulation of the central and lateral abdominal muscles. This programme alternates more rapidly than the previous one, with series of 60-Hz impulses alternating every second without pauses. A complete programme recommended for toning, definition and strength training.

PROG. 15 - toning, definition and strength training - duration 30 minutes.

An advanced programme, with even faster alternation. The series of 60-Hz impulses alternate every half second; ideal for intense and targeted toning, increasing strength and definition.

TROUBLESHOOTING GUIDE

Problems	Possible Causes	Solutions
A beep is heard and a message appears on the display to check the belt's adhesion to the skin.	The belt is not properly adhered to the skin.	Tighten the belt to ensure optimal contact with the skin.
	The control device is not correctly positioned on the belt.	Ensure that the control device is properly attached to the belt in the correct direction using the magnets.
	The conductive pads and skin are not sufficiently moistened.	Moisten both the conductive pads and the skin to improve conduction.
	The presence of dust deposits or oxidation on the metal contact pins between the belt and the device may prevent electrical contact.	Use a stiff brush, preferably metallic, to carefully clean the metal buttons, especially in the recessed area where contact occurs. Then, blow to remove any remaining debris.
		
The display works, but the belt does not generate stimulation.	The intensity may be set too low.	Press the "+" button to increase the intensity and check that you are adjusting the level of the currently used channel.
Contractions are very weak even when the intensity is high, and discomfort is felt.	The conductive pads and skin may not be sufficiently hydrated.	Moisten both the conductive pads and the skin to improve conduction.
The display does not turn on.	The battery is discharged.	Recharge the battery.
Issues different from those listed above.	The belt or one of its components may be defective.	Contact TESMED customer support at info@tesmed.com .

NOTES REGARDING THE BATTERY

- The device is powered by a rechargeable battery. The battery can be charged and drained hundreds of times, but its lifespan is not unlimited. In order to guarantee optimal performance, use the device regularly.
- Typical battery charge time (if drained to 0%): 90 minutes.
- After 3 hours of charging, disconnect the device from the charging source even if it is not charged.
- Maximum ambient temperature for device charging 30 °C.
- If the device overheats while charging, disconnect the device from the charging source immediately.
- If the device appears to be broken, cracked or damaged externally, do not charge it.
- In order to save energy, disconnect the USB cable from the control unit when the battery is fully charged.
- Leaving the battery in hot or cold environments, for example in a car during the summer or winter, will reduce the capacity and the lifespan of the battery. A device with a hot or cold battery may momentarily fail to function, even when the battery is charged. Battery performance is particularly limited in temperatures much below freezing.
- Do not throw the battery into the fire as it may explode. The battery may also explode if damaged. Dispose of batteries in accordance with local regulations. Recycle whenever possible. Do not dispose of in domestic waste.

DISPOSAL

At the end of the product lifecycle, do not throw this product or batteries into the normal household garbage, but bring it to a collection point for the recycling of electronic equipment. Some product materials can be re-used if you bring them to a recycling point. By re-using some parts or raw materials from used products you make an important contribution to the protection of the environment. Please contact your local authorities if you need more information about collection points in your area. Waste electrical and Electronic Equipment can have potentially harmful effects on the environment. Incorrect disposal can cause harmful toxins to build up in the air, water and soil and can be harmful to human health.

TECHNICAL CHARACTERISTICS OF ABSOLUTE	
CLASSIFICATION	Device with internal power source
INTENDED USE	Muscular abdominal stimulation
POWER SUPPLY	Rechargeable 3.7 V, 500 mAh lithium-polymer (LiPo) battery
PULSE REPETITION FREQUENCY	2 -150 Hz
IMPULSE BREADTH	80-325 uS
MAXIMUM ENERGY DELIVERED PER PULSE	3.50 mJ/impulse
MAXIMUM PEAK POWER	5.4 W/impulse
MAXIMUM CONTINUOUS POWER	550 mW
AVERAGE OUTPUT CURRENT AT MAX. FREQUENCY	10 mA
AVERAGE OUTPUT CURRENT AT MIN. FREQUENCY	70 µA
MAX. PEAK OUTPUT CURRENT	110 mA
PULSE SHAPE	Square symmetrical biphasic waveform
BELT LENGTH	120cm
CHANNELS	2 independent and individually adjustable channels, each insulated electrically from the other
ALL THE PARAMETERS SHOWN IN THE TABLE REFER TO EACH INDIVIDUAL CHANNEL, WITH A TYPICAL CHARGE OF 500 OHM	
BATTERY RECHARGING	
INPUT POWER SUPPLY	5Vdc-500 mA
RECHARGING CABLE	USB 2.0 with Type-C connector
THE BATTERY CANNOT BE SUBSTITUTED BY THE USER	

CONTACTS

Information on use and technical assistance:

info@tesmed.com

+39 035 662660

every working day from 8.30 am to 12.30 am and from 1.00 p.m. to 5.00 pm

**WhatsApp Tesmed
Fast Check Video**

If appropriate, it is possible to send a short video concerning the device, to demonstrate any operating problems - the message must be accompanied by a brief written explanation, with a screen of the order number and the date of purchase. Service active on working days and hours.

+ 39 345 6979093

A copy of the manual in electronic format can be obtained on the tesmed.com website at the INFORMATION AND MANUALS page by selecting Manuals

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Feldi S.r.l. - Italy