INTIMINA™

[INTIMINA.COM/SUPPORT](https://www.intimina.com/support)

EN GETTING STARTED: This booklet contains basic instructions; for complete details, visit INTIMINA.COM/SUPPORT and download the full manual.

REGISTER WARRANTY: To activate your warranty for Intimina device, please contact Customer Care at customer-care@intimina.com. Keep your original purchase receipt as evidence that the date of your claim is within the warranty period.

1-YEAR WARRANTY (ELECTRIC DEVICES ONLY): INTIMINA warrants electric devices for a period of ONE (1) YEAR after the original date of purchase against defects due to fault in workmanship or materials arising from Normal Use of the device. The warranty covers working parts that affect the function of the device. It does NOT cover cosmetic deterioration caused by fair wear and tear, or damage caused by accident, misuse or neglect. Any attempt to open or take apart the device (or its accessories) will void the warranty. If you discover a defect and notify INTIMINA during the warranty period, INTIMINA will, at its discretion, replace the device free of charge. Claims under warranty must be supported by reasonable evidence that the date of the claim is within the warranty period. To validate your warranty, please keep your original purchase receipt together with these warranty conditions for the duration of the warranty period. To claim your warranty, you must log in to your account at INTIMINA.COM then select the option to make a warranty claim. Shipping costs are nonrefundable. This undertaking is in addition to your statutory rights as a consumer and does not affect those rights in any way.

BATTERIES / CHARGING: Personal massagers (Kiri, Raya, Celesse) require 2 AAA batteries. KegelSmart 2 and Kalia are rechargeable with included USB cable. Before charging ensure DC socket is free from liquid.

WARNINGS: Consult your healthcare provider prior to using INTIMINA products if you have gynecological or medical concerns including sexually transmitted infection, pelvic infection, any inflammation of the pelvic area. Always wash hands and products before and after each use. Consult your healthcare provider before use if you: are pregnant or have given birth within the last 12 months; have an IUD, an implanted device, or have had surgery on your intimate areas. Do not use if you are allergic to silicone. INTIMINA products are not contraceptives and will not prevent against pregnancy and/or sexually transmitted diseases. Discontinue use and contact a healthcare provider if you experience any pain or discomfort. Keep out of reach of children and animals. Do not expose to open flames. Do not use harsh cleaners, petroleum based substances, tea tree oil, alcohol, or bleach to clean your products. **Toxic Shock Syndrome (TSS):** do not use a menstrual cup if you have had TSS. TSS is a rare but serious disease that may be fatal and has been linked to high absorbency and prolonged use of tampons. Symptoms can include sudden high fever, vomiting, diarrhea, dizziness, fainting, and a rash like sunburn. If you have any of these signs while using a menstrual cup, remove it and seek immediate medical attention.

HOW TO USE: MENSTRUAL CUPS: Lily Cups and Ziggy Cup are intended to be used as reusable receptacles that are inserted into the vagina to collect menstrual fluid. **Lily Cups:** simply fold the cup, insert fully into the vagina then squeeze the base to open the rim and form a seal. Wear for up to 8 hours. To remove, squeeze the base to release the seal, slide out, rinse, and reinsert. Do not use during intercourse.

Ziggy Cup and Ziggy Cup 2: squeeze the rim together, insert fully into the vagina and tuck behind the pubic bone. Wear for up to 8 hours, then remove by hooking your finger behind the rim, rinse and reinsert. Can be worn during intercourse. **PELVIC FLOOR STRENGTHENING: KegelSmart 2:** a smart Kegel trainer that is intended for independent use by adult women to strengthen the pelvic floor. Turn on by pressing the (O) button then insert into the vagina, leaving the retraction cord outside the body. Contract when it vibrates and rest when it stops. 3 quick vibrations will signal the end of the routine, and the device will turn off automatically. Remove using retraction cord. To manually turn off, hold the (O) button for several seconds.

Laselle: intended for independent use by adult women to strengthen pelvic floor muscles. Insert into the vagina, leaving the retraction cord outside the body. Contract your vaginal muscles, lifting up and in. Hold contraction for 2-10 seconds, then relax for the same amount of time and repeat. Daily use should not exceed 60 minutes. Remove using retraction cord. To link exercisers, thread the retraction cord of the first ball through the outer ring of the other. Pass the first ball back through the resulting loop and pull tight. Do not boil or open the balls. **PERSONAL MASSAGERS: Kiri, Raya, Celesse:** to turn on and increase vibration strength, press and hold the (+) button. Press (O) to cycle through the stimulation modes. To reduce the intensity or turn off, press the (-) button. **Kalia:** press the button twice to turn on and once to switch vibration modes. Press and hold to turn off.

DISPOSAL INFORMATION: Disposal of old electric equipment (applicable in the EU and other European countries with separate waste collection systems). The crossed-out dustbin symbol indicates that this device should not be treated as household waste, but rather be brought to the appropriate collection point for the recycling of electrical and electronic products. Please see back page for instructions on how to remove the battery.

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© 2021 LELOi AB All rights reserved.
Suzhou Armocon Technology Co., Ltd.
LELOi AB, Karlavägen 41, 114 31 Stockholm, Sweden
EU Distributor: LELOi AB, Karlavägen 41, 114 31 Stockholm, Sweden
US Distributor: LELO Inc., 5799 Fontanos Way, San Jose, CA 95138, USA