THANK YOU FOR JOINING  
THE LUCENT FAMILY


Running low on your whitening supplies? No worries!

Refilling your home teeth whitening kit is just a click away.


Visit our website for a hassle-free restock of your whitening essentials.

Keeping your smile bright has never been easier!

Happy whitening!

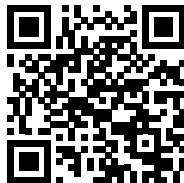BE  
LUCENT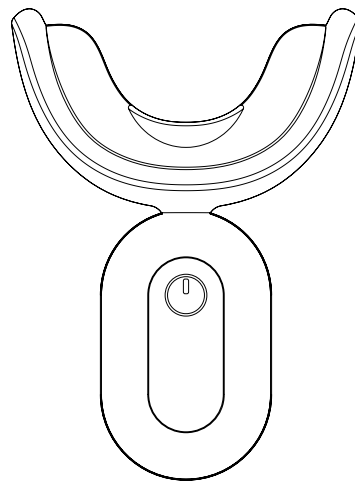

STELLAR

Gentle Teeth Whitening Kit

Welcome to a brighter,  
more confident smile!

[www.be-lucent.com](http://www.be-lucent.com)  
[info@be-lucent.com](mailto:info@be-lucent.com)  
[@be\\_lucent\\_now](https://www.instagram.com/be_lucent_now)


We're thrilled to have you embark on this journey with us. At Be Lucent, we believe that everyone deserves to shine, starting with a radiant smile. That's why we've crafted this home teeth whitening kit with you in mind - to make professional-level dental care accessible right from the comfort of your home.

Our passion is to innovate and enhance everyday dental routines, transforming them into something extraordinary. With a blend of cutting-edge technology and the finest ingredients, we create products that are not only effective but also safe and easy to use. This kit is a testament to our commitment to excellence, designed to help you achieve visible results while prioritizing the health of your teeth and gums.

Whether you're preparing for a special occasion or simply want to add a spark to your daily life, our teeth whitening kit is the perfect companion to take your dental routine to the next level. Let's embark on this journey towards a dazzling smile together!

Thank you for choosing us to be a part of your smile journey.


## OVERVIEW

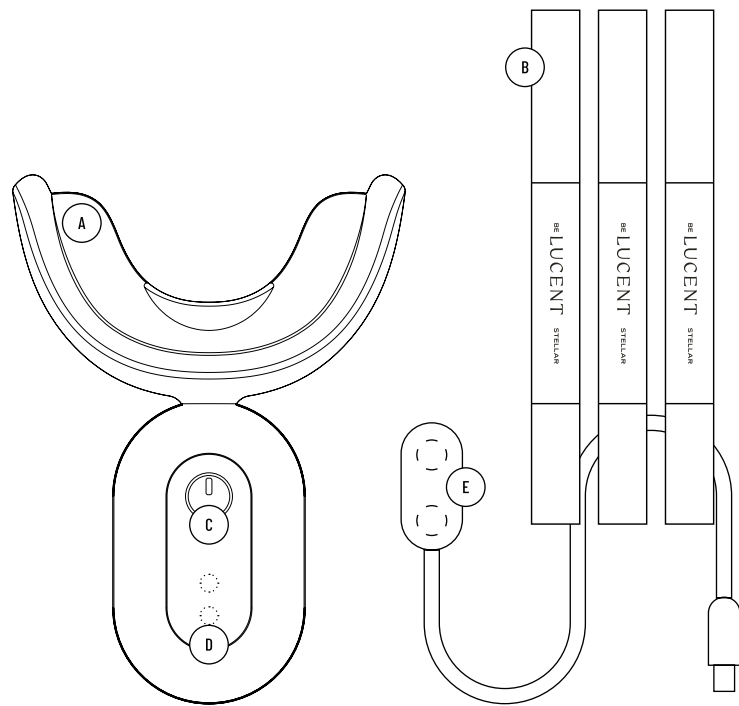1 Brush your teeth and dry them with a tissue

2 Apply gel on the teeth evenly, avoid the gums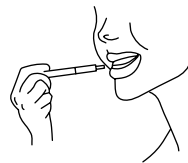

3 Put the activator in your mouth and bite on it gently

4 Press the power button for 20 minutes treatment

5 Rinse the mouth thoroughly with water

6 Rinse the mouth tray after each use

## STELLAR

Gentle Teeth Whitening Kit

Smile Brighter!

# STELLAR

## Gentle Teeth Whitening Kit

|                            |             |       |
|----------------------------|-------------|-------|
| INSTRUCTIONS OF USE        | ENGLISH     | 2-3   |
| HANDBUCH                   | DEUTSCH     | 4-5   |
| INSTRUCTIONS D'UTILISATION | FRANÇAIS    | 6-7   |
| MANUAL DE USARIO           | ESPAÑOL     | 8-9   |
| ISTRUZIONI D'USO           | ITALIANO    | 10-11 |
| GEBRUIKSAANWIJZINGEN       | NEDERLANDS  | 12-13 |
| INSTRUKCJA UŻYCIA          | POLSKI      | 14-15 |
| BRUKSANVISNING             | SVENSKA     | 16-17 |
| BRUGSANVISNING             | DANSK       | 18-19 |
| BRUGSANVISNING             | NORSK       | 20-21 |
| KÄYTTÖOHJEET               | SOUMALAINEN | 22-23 |

## Overview

- A. LED lights in blue and red
- B. Tubes of PAP+ whitening gel
- C. Touch button
- D. Charging with magnetic connection
- E. Magnetic charging cable

## INCI

Glycerin, Aqua, Phthalimidoperoxycaproic Acid, PVP, Ammonium Acryloyldimethyltaurate/VP Copolymer, Potassium Hydroxide, Potassium Citrate, Hydroxyapatite, Mentha Piperita (Peppermint) Oil, Mica, CI 77891, Saccharin Sodium, Proctone Olamine

## Introduction

### Thank you for buying the Stellar whitening kit.

The whitening kit includes the PAP+ gel, which cleans and whitens the teeth. But it's also gentle on the gum and mouth. The formula is also developed to support the strengthening of the teeth after treatment.

The activator is equipped with LED lights, using blue and red light, with the wavelength between 480-520 nm. This activates the gel and the process cleans the teeth from stains and marks and turns yellow teeth whiter.

### Important notes

The first time you might want to use only 10 minutes to get used to the process.

Try to avoid smoking and drinking coffee, tea or red wine for the following hours after treatment. The teeth are more sensitive to discoloring right after the treatment.

## Precautions

The whitening process does not work on artificial teeth nor teeth that are damaged.

The whitening kit should not be used by pregnant and breastfeeding women nor children under 16 years

The silicon rail on the activator is waterproof and should be cleaned under tap water. But the main body of the activator is not waterproof and must not be immersed under water.

Do not pour boiling water on the silicon rail, as it might damage the shape.

Avoid any contact with the eyes.

The saliva produced during the whitening process is ok to swallow.

If you would experie irritation or swollen gum, please discontinue the program and contact a dentist.

Do not use the whitening kit while sleeping.

Keep the kit away for children under 3 years old.

Do not try to detach, deform nor repair the device yourself.

## Quick guide

The whitening process should be repeated for about 7 days. Some people need a few more days to reach the desired whitening effect.

Refill packs are available at [be-lucent.com](http://be-lucent.com)

### 1. Prepare your teeth

Brush your teeth to make sure they are clean. Then dry your teeth with a tissue or towel.

### 2. Apply the gel

Unscrew the lid of the teeth whitening gel. Rotate the bottom of the tube clockwise until the gel flows into the brush. Then paint the gel onto your teeth. Add extra gel on yellow teeth. Avoid putting gel on the gums.

### 3. Prepare the whitening activator

Place the mouth tray into your mouth and bite gently. Find the best fit for your teeth. Then press the button to turn the light on.

### 4. Use the whitening activator

The process runs for 20 minutes and will automatically turn off when it's done. The light will also pause for half a second after 10 minutes to inform about half-time.

### 5. Rinse in water

Rinse your mouth with water after taking the activator out. Clean the mouth tray under tap water.

## One year limited warranty

All Be Lucent products are designed and manufactured to meet high expectations and to provide the best customer experience. However, sometimes, products may have malfunctions due to manufacturing defects. That is why we offer a full one year warranty from the date of original purchase, which covers defects in materials and workmanship.

The replacement warranty is valid only if the original receipt issued to the customer, specifying the date of purchase, is presented with the product to be replaced.

If you believe this product is defective, carefully repack the unit and return the product to your authorized seller within the warranty period, with the original receipt. Your authorized seller will replace the product, provided a defect in materials or workmanship is identified.

This limited replacement warranty does not apply in cases of abuse or misuse of the product, use contrary to the Lucent instructions, ordinary wear and tear, incorrect connection, force majeure or unauthorized repair.

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## INCI

Glycerin, Aqua, Phthalimidoperoxycaproic Acid, PVP, Ammonium Acryloyldimethyltaurate/VP Copolymer, Potassium Hydroxide, Potassium Citrate, Hydroxyapatite, Mentha Piperita (Peppermint) Oil, Mica, CI 77891, Saccharin Sodium, Piroctone Olamine

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