# Carepet

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## Articoli inclusi

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
USER MANUAL

## Items included

1. Arctic velvet and suede fabric cover
2. Orthopaedic mattress with waterproof cover
3. Smart bed frame upholstery
4. Smart sleep monitor
5. USB charger
6. User guide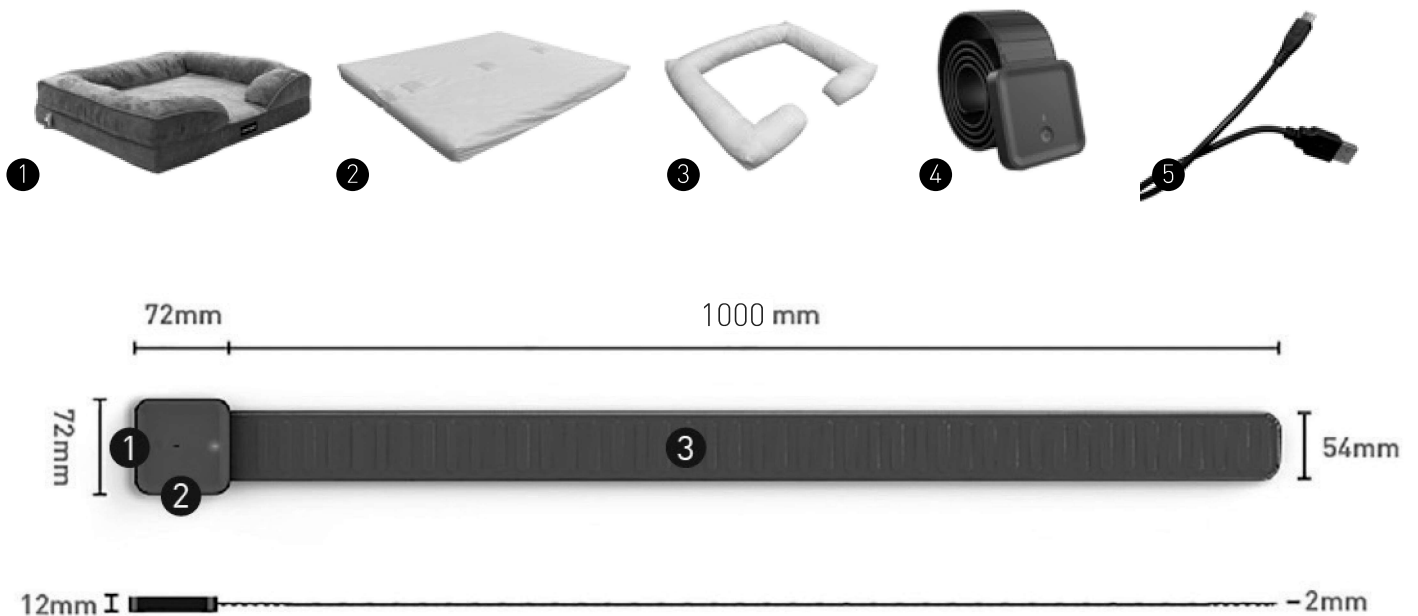

1. Mini USB for charging
2. Bluetooth transmitter
3. Sensor headband

## What is Carepet

Carepet is the smart bed designed to monitor your pet's sleep quality and well-being. This contactless device collects and analyses data on your pet's sleep thanks to an extremely sensitive internal sensor, capable of detecting even the faintest body movements. The signals are processed using a high-precision digital amplification technique, combined with accurate simulations and digital filters, to extract information on breathing and heartbeat during sleep. Carepet provides a daily assessment of sleep quality, analysing your pet's different sleep states and patterns. In addition, it offers customised suggestions on the ideal rest time and wake-up times, based on weekly sleep monitoring.

## Preparing to use your Carepet

1. To save battery power, the sleep monitor is in hibernation mode when it leaves the factory. Please charge the device with the USB cable to activate it. While charging, the red light will flash.
2. Make sure the device has enough battery power and then insert the sleep monitor into the mattress via the loops provided.
3. Insert the mattress inside the fleece structure.
4. Have the pet lie on Carepet to automatically track sleep data.

# 1. Download the App

Go to the Apple App Store or Google Play Store, and search for 'Carepet' to download our 'Carepet' app.

Or you can also scan the QR code and download the app directly to your iOS or Android phone.

Make sure that Bluetooth is active on your smartphone. Important: check that your Android operating system is version 4.3 or higher and your iOS version is version 7.0 or higher, and that your device has Bluetooth 4.0.

## 2. Battery charging

2.1 Press the device button; if the device's red light flashes twice, it indicates a low charge and you need to charge it.

If the device light is blue, it means there is enough battery.

2.2 During charging, the red light will flash.

2.3 When the battery is fully charged, the red light will stop flashing and remain lit.


Please view the Carepet status section to check the battery level. For the first charge, we suggest charging for more than 2 hours.

**Do not use the sleep monitor while charging.**

### 3. Sensor design

Before use, please place the sensor in the mattress via the loops. The sleep monitor has a super-sensitive sensor that will automatically track your health and sleep data using weak body movement signals, without the need to connect your phone. Sleep data can be stored in the device for 30 days and can be retrieved at any time according to your actual needs.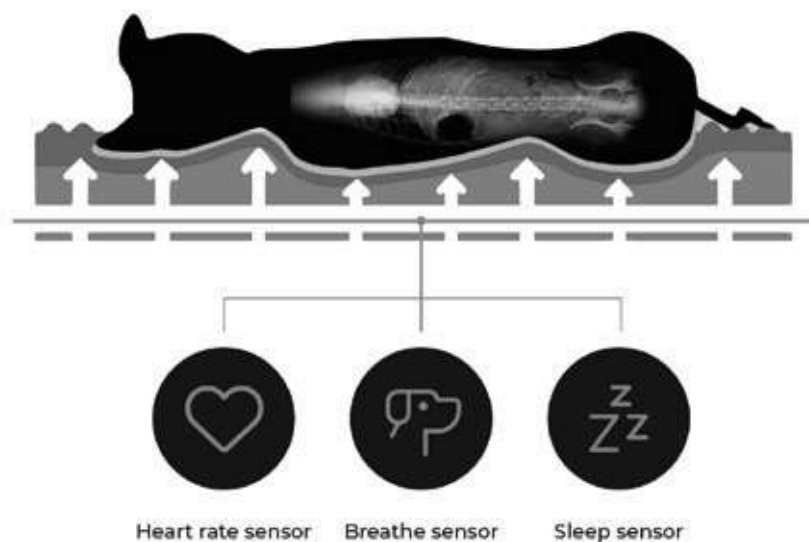


## 4. Device connection

Make sure you have Bluetooth on your phone.

Step 1: Open the Carepet app

Step 2: Press the + button to start searching for the device

Step 3: Press the available Carepet to start the connection

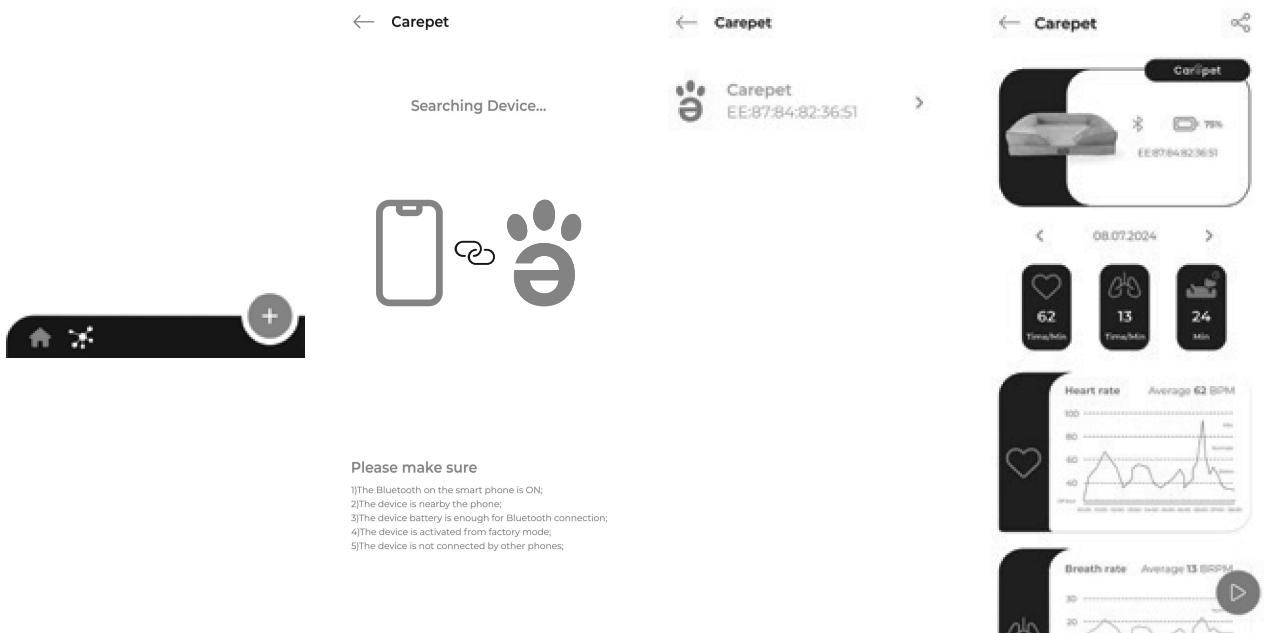

## 5. Real-time monitoring

After connecting the device, press the play button to start real-time monitoring of sleep quality.


## 6. Warning of abnormal heart rate and respiration rate

Users can set the abnormal value of heart rate and respiration rate, and the app will sound an alarm when one of these values is abnormal.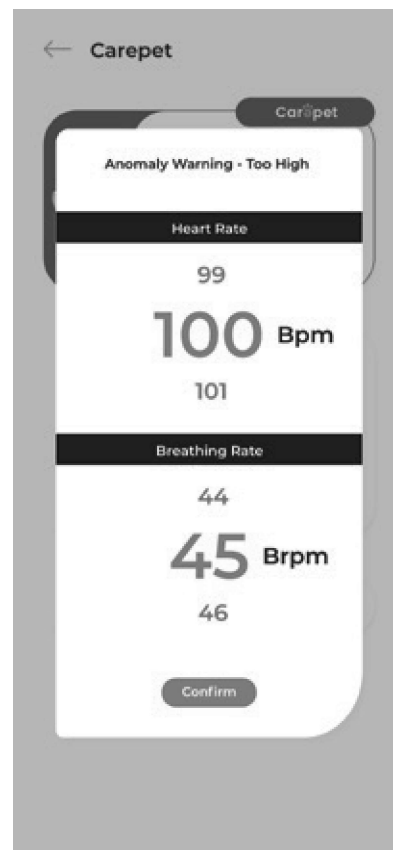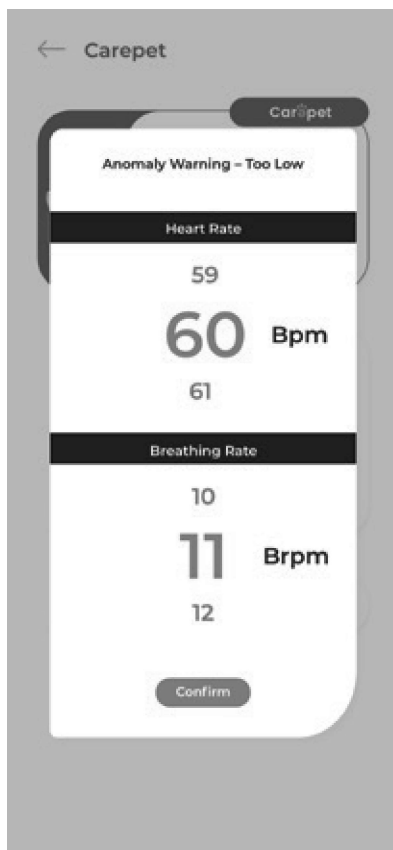

# 7. Sleep quality report

Go to the 'Reports' page to check the detailed daily, weekly and monthly sleep quality analysis report.

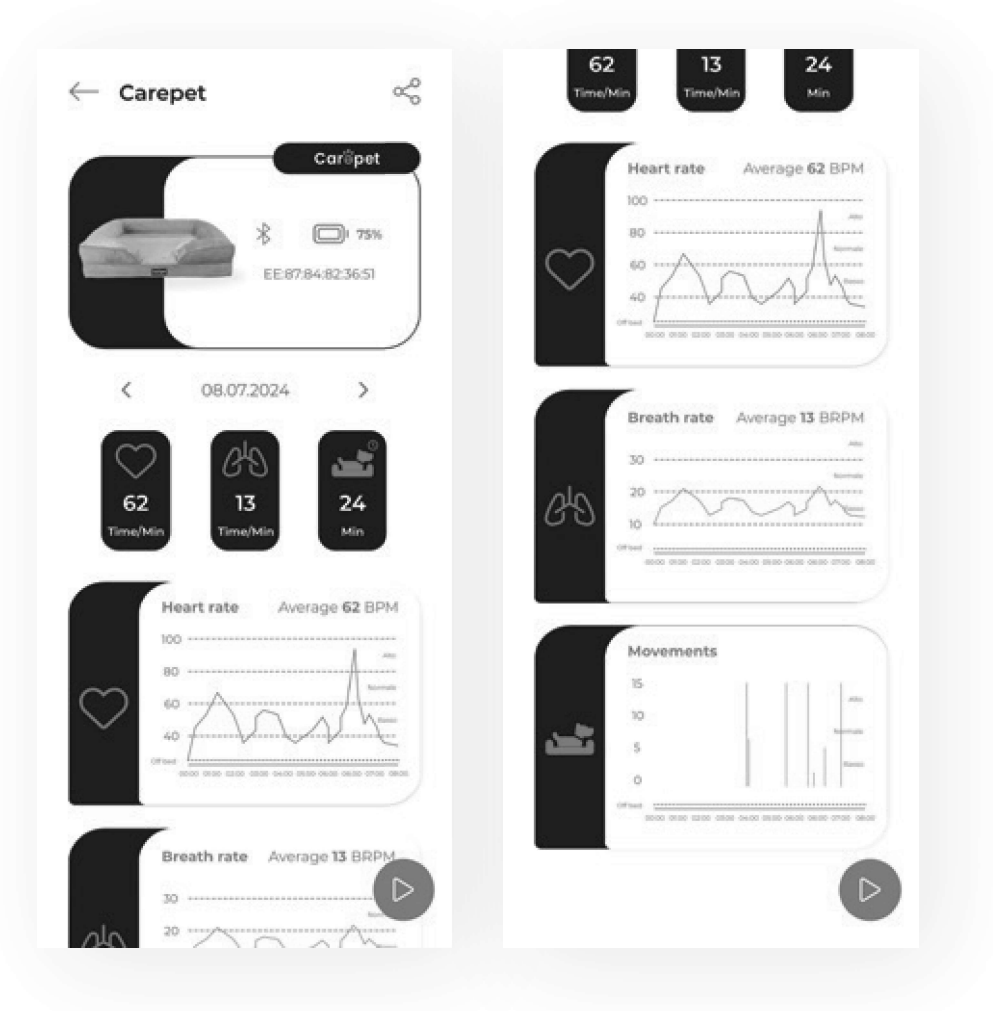## Parameters:

Product name: Carepet

Connection method: BLE (Bluetooth 4.0)

Power supply: Rechargeable lithium battery 4.2 V capacity 350 mAh.

Device dimensions: 88mm × 63mm × 16mm

Sensor dimensions: 1000mm × 53mm × 3.5mm

Operating temperature: -20°C to 65°C

Charging current: 5.0V - 500mA

## Important Notice

Note: The sensor tape is very sensitive and requires careful protection. The sensor cannot be bent when lifting. Do not bend at a tight angle, otherwise the bent sensor part will affect the sensor signal and compromise the accuracy of the sleep monitor.

## Care and Maintenance

1. Sleep monitor: When washing, separate Carepet from the sleep monitor so as not to damage the device by losing the warranty. The sleep monitor is not a medical device and is not designed to diagnose or treat any disease. For the diagnosis and treatment of other diseases, please consult a veterinary doctor.

If necessary, place the intelligent sleep monitor in a ventilated environment, clean it with a wrung-out damp cloth, removing dust or stains from the surface.

2. When washing arctic velvet and suede fabrics, it is recommended to keep the water temperature at about 30°C. Too high a temperature can cause the fabric to shrink and deform, as well as damage the lint on the surface of the fabric, worsening its texture.

Use lukewarm water at about 30°C and add a suitable detergent. It is also not appropriate to use a tumble dryer to dry arctic velvet and suede fabrics. Drying at high temperatures can cause the suede fabric to shrink and harden, and the lint can be damaged by the high temperature, causing the fabric to lose its original lint. It is best to lay the products in a well-ventilated place to dry naturally.

## Security Information

When using this device, always follow the basic safety precautions listed below to reduce the risk of accidents, electric shock, injury and fire.

1. Switch off the device near chemical plants, petrol pumps, and other places containing explosive objects.
2. Exercise caution when using Carepet in the vicinity of devices such as pacemakers, audio prostheses, and other electromedical implants that may receive interference from the mobile phone radio signal.
3. In the presence of a pacemaker, the monitor's Bluetooth transmitter should be kept more than 15 cm away from the heart area.
4. Never disassemble the sleep monitor yourself.
5. Charge in a well-ventilated environment and keep away from flammable and highly explosive sources.
6. To avoid demagnetisation, keep the device away from hard disks, credit cards and other objects that work with magnetic fields.
7. Unless otherwise stated in the manual, keep the device away from liquids. If it falls into water, do not attempt to turn it on again until it has been thoroughly dried and you are sure there is no moisture inside.
8. Avoid using the device in extremely low or high temperature environments. Never leave the device in outdoor environments with direct sunlight, high humidity or full of dust.
9. Do not use liquids or strong detergents to clean the sleep monitor.

## **Sleep Monitor Battery Warnings**

The product is equipped with a battery that may explode or catch fire if mishandled. To avoid exploding or igniting the battery, do not attempt to open or repair it. Do not disassemble, bump, puncture, short-circuit, or dispose of the battery in a fire. Do not drop it in water or expose it to temperatures above 60 degrees. Do not attempt to replace the battery. Do not use sharp objects to remove the battery. Keep it out of the reach of children.

## **Attention**

This is not a medical device. The measurements taken by Carepet are purely indicative and in no way represent measurements with medical relevance and cannot be used for diagnostic purposes. For precise measurements, we recommend consulting a veterinarian and using professional medical devices.

# M

8 052693 667024  
Carepet smart bed

8 052693 667000  
Smart sleep monitor

# L


8 052693 667031  
Carepet smart bed

8 052693 667000  
Smart sleep monitor

# XL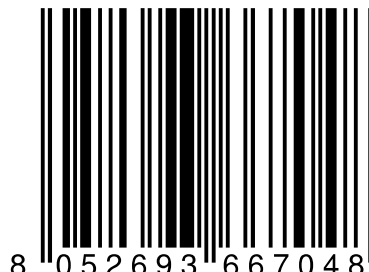

8 052693 667048  
Carepet smart bed

8 052693 667000  
Smart sleep monitor

Made in China  
Imported by Domethics s.r.l.  
Via Pietro Micca 20 10122  
Torino, Italy [contact.us@domethics.com](mailto:contact.us@domethics.com)

PAP

PS

LDPE